# AeroDawg

# AeroKat

Canine Aerosol Chamber

www.aerodawg.com

Trudell Medical International  
725 Baransway Drive, London, Ontario, Canada, N5V 5G4  
+1-519-455-7060 +1-519-455-7858  
info@trudellmed.com  
www.trudellmed.com

Trudell Medical International

Feline Aerosol Chamber

www.aerokat.com

101973-001 Rev. B \* Trade marks and registered trade marks of Trudell Medical International (TMI). Copyright © TMI 2019. All rights reserved.

## EN INTENDED USE

The chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to your cat or dog's lungs as prescribed by your veterinarian.

## BEFORE USE

The chamber can be used directly out-of-package. Ensure these instructions and the instructions supplied with the inhaler have been read. Carefully examine the chamber and replace immediately if any defect is noticed. The chamber is shipped with 2 different sized masks. Select the mask that fits best and insert on frontpiece of chamber. The mask should cover the animal's nose and mouth but not eyes.

## USAGE INSTRUCTIONS

**1** Remove cap and shake the inhaler immediately before use as per the instructions supplied with it. **2** Insert the inhaler into the backpiece of the chamber and gently apply the mask to your animal's nostrils while gently holding the muzzle closed. **3** Press the inhaler to release the medication. Hold the mask in place for 5-6 breaths as the cat or dog inhales the medicine.

## TIPS AND TRICKS:

- Slowly introduce the chamber. Start with the mask only, getting the animal used to the feel of it
- Reward the animal with treats before and / or after treatment
- Swaddle them in a towel or blanket
- If your cat or dog does not respond well to the sound of the inhaler, hold the chamber behind your back and press the inhaler and then immediately place mask on their face.

## Notes:

- Use the **Flow-Vu**® Inhalation Indicator to ensure a good seal, coordinate pressing the inhaler with inhalation and to count the number of breaths taken. The **Flow-Vu**® Indicator moves toward your cat or dog as they inhale and only moves if there is a proper seal.
- Administer one (1) puff at a time. Follow instructions supplied by your veterinarian on the number of puffs to take and how long to wait before repeating.
- Wipe the animal's face with a damp cloth after treatment to remove medication residue.

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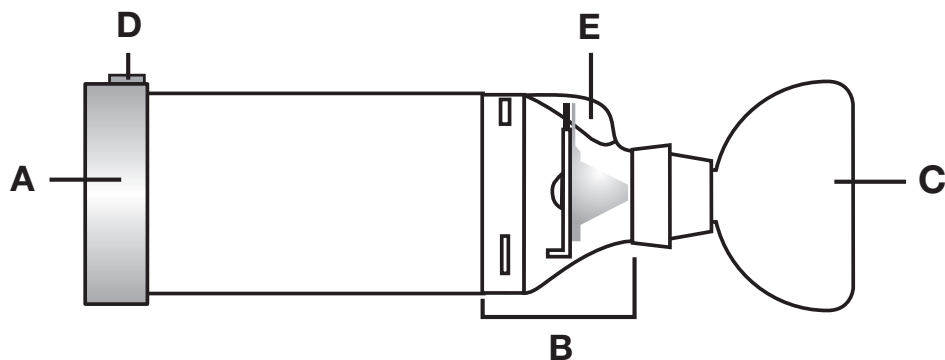
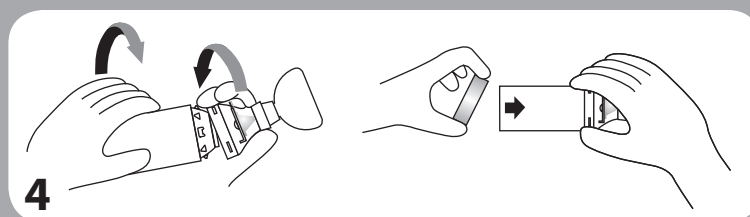
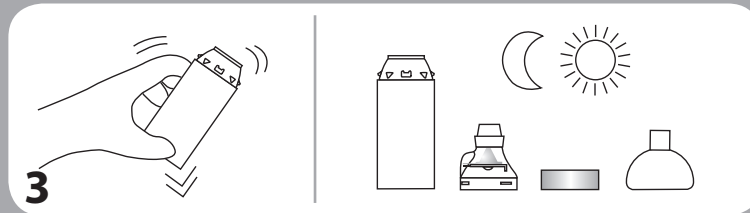
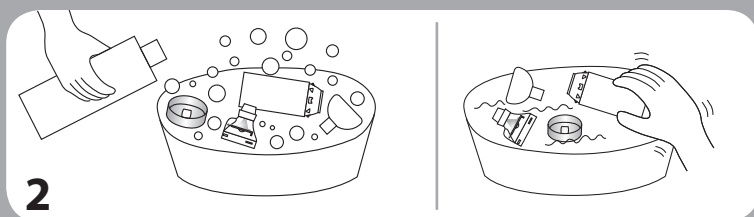
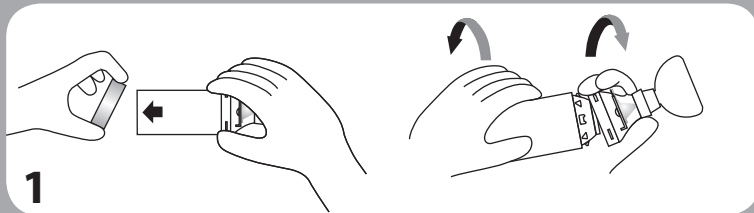
## EN CLEANING INSTRUCTIONS

The chamber should be cleaned weekly.

**1** Remove the backpiece (A). To detach the frontpiece (B), twist chamber as shown and then remove mask (C) from frontpiece. **2** Soak the parts for 15 minutes in a mild solution of liquid dish detergent and lukewarm clean water. Agitate gently and then rinse parts in clean water. **3** Shake out excess water and allow to air dry in a vertical position. Ensure parts are dry before reassembly. **4** To reassemble, fit the mask on the frontpiece and then place frontpiece on the end of the chamber and twist firmly until securely locked into position. Center the Alignment Feature (D) on the backpiece with the **Flow-Vu\*** Indicator (E) as shown. Press firmly to attach the backpiece.

### Notes

- If you notice medication build-up in your chamber, wash the inside of the chamber gently with a soft cloth.
- Do not boil, sterilize or clean in a dishwasher.
- Product should be replaced after 12 months of continuous use

### Cautions

- Do not leave the chamber unattended with children.

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